Rapid Tests Training Guide

How to Administer Test

All persons administering a test must be trained to do so according to the province in which they are performing the test. For further details, please refer to your provincial ministry of health.

1. Wear appropriate PPE, as outlined by your local provincial ministry of health (e.g., gloves, apron, mask, sneeze guard, etc.). Ensure testing area has been appropriately disinfected before each test. Open test.
2. Hold buffer bottle vertically and fill extraction tube with fluid until fill line. Place the extraction tube in tube rack.
3. Swab nose. Tilt head backwards (approx. 70°). Insert swab 2 cm into

nostril and rotate swab 5 times against nasal wall. Repeat in other nostril with the same swab.

4. Insert used swab into extraction tube. Swirl swab on the sides of the extraction tube 5 times and squeeze out swab by holding the sides of the extraction tube with your fingers tightly. Break swab at breakpoint. Place cap on extraction tube and close tightly.
5. Open nozzle cap at the bottom of the extraction tube. Dispense 5 drops of extracted specimen onto the device. Leave test unmoved until ready to read (approx. 15-20 minutes). Close nozzle and begin disposal procedure.

For video instructions, please refer to Abbott video: <https://alere.wistia.com/medias/82co9bznbcb>

How to Dispose of Test


1. Ensure both caps on the extraction tube are sealed tightly.
2. Dispose of both the extraction tube (with swab inside) and testing device into an approved biomedical waste disposal bin.
3. Clean area with disinfectant and follow proper doffing and disposing procedures for PPE. Ensure new PPE is used for each test procedure and patient.

For detailed disposal instructions, please refer to your local provincial health ministry website. Please note the Province of Ontario requires additional disposal steps, found here: <https://covid-19.ontario.ca/provincial-antigen-screening-program>

How to Interpret Test Results


Negative

The presence of only the control line (C) and no test line (T) within the results window indicates a negative test result.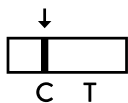


Positive

The presence of the control line (C) and test line (T) within the results window indicates a positive test result, regardless of which line appears first, or how faint. **Health Canada recommends getting a lab-based PCR test to confirm positive results.** Check your local public health unit for testing locations.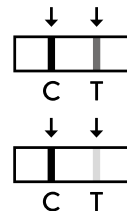

Invalid

The presence of no lines within the results window, or the presence of only the test line (T) indicate an invalid test result. Reread instructions on administration before retesting the specimen with a new testing device.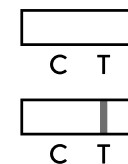

For full details, please refer to the Abbott leaflet found inside the testing kit, and here:

https://extranet.who.int/pqweb/sites/default/files/documents/EUL_0587_032_00_PanbioCOVID-19_AgRapidTestDevice_NASAL_ifu.pdf


For reporting instructions and requirements for positive cases, please refer to your local provincial ministry of health website.

Additional training resources: Some provinces also have specific guidance and training material on rapid testing, see: <https://www.canada.ca/en/public-health/services/diseases/coronavirus-disease-covid-19/testing-screening-contact-tracing/workplace/rapid-antigen-tests.html#a3>


TEST PROCEDURE

1 Hold the buffer bottle vertically and fill the extraction tube with buffer fluid until it flows up to the Fill-line of the extraction tube (300µl).

⚠ Caution: If the amount of buffer is excessive or insufficient, an improper test result may occur.

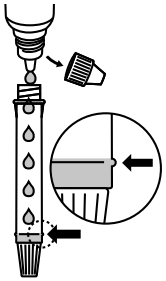
2 Place the extraction tube in the tube rack.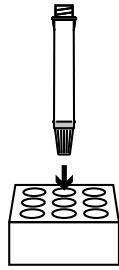


3 Tilt the patient's head back 70 degrees. While gently rotating the swab, insert swab less than one inch (about 2 cm) into nostril (until resistance is met at the turbinates). Rotate the swab five times against the nasal wall. Using the same swab repeat the collection procedure with the second nostril. Slowly remove swab from the nostril.


⚠ Caution: If the swab stick breaks during sample collection, repeat sample collection with a new swab.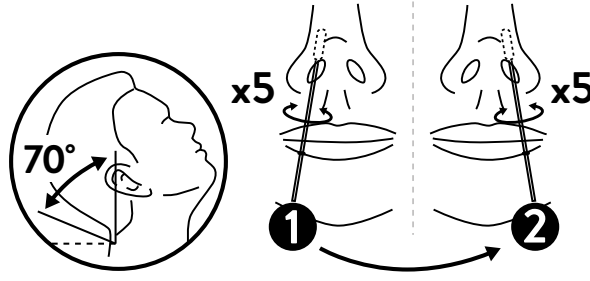


4 Insert the swab specimen in the extraction tube. Swirl the swab tip in the buffer fluid inside the extraction tube, pushing into the wall of the extraction tube at least five times and then squeeze out the swab by squeezing the extraction tube with your fingers.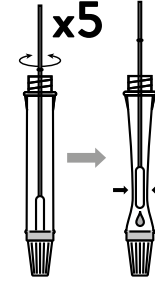

5 Break the swab at the breakpoint and close the cap of extraction tube.

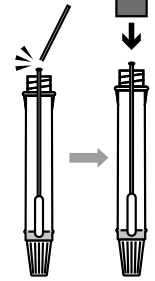
6 Open the dropping nozzle cap at the bottom of the extraction tube.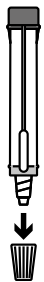


7 Dispense 5 drops of extracted specimens vertically into the specimen well (S) on the device. Do not handle or move the test device until the test is complete and ready for reading.

⚠ Caution: Bubbles that occur in the extraction tube can lead to inaccurate results. If you are unable to create sufficient drops, this may be caused by clogging in the dispensing nozzle. Shake the tube gently to release the blockage until you observe free drop formation.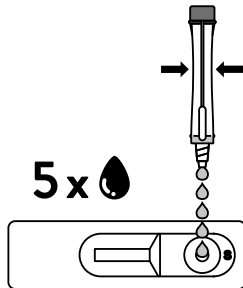

8 Close the nozzle and dispose of the extraction tube containing the used swab according to your local regulations and biohazard waste disposal protocol.

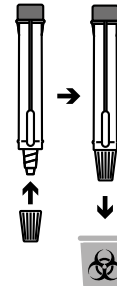9 Start timer. Read result at 15 minutes. Do not read results after 20 minutes.

10 Dispose of the used device according to your local regulations and biohazard waste disposal protocol.

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